medi GmbH & Co. KG
MedicusstraBe 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T +32-11 24 25 60
 F +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Itapeva 538 cj. 42
 Bela Vista, São Paulo - SP
 CEP: 01332-000
 Brazil
 T +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Orient M.E.A. FZC
 TAURUS GROUP
 Suite No. M01 Wasel Al Mamzar Bldg.
 Al Wuhida Road, Al Mamzar
 P.O.BOX NO. 91464
 DUBAI, UAE
 T: +971 4 2557344
 F: +971 4 2557399
 md@taurusgroup.net

medi Polska Sp. z o.o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi UK Ltd.
 Plough Lane
 Hereford HR4 OEL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

medi Ukraine LLC
 Tankova Str. 8, office 35
 Business-center „Flora Park“
 Kiev 04112
 Ukraine
 T +380 44 591 11 63
 F +380 44 392 73 73
 info@medi.ua
 http://medi.ua


medi USA L.P.
 6481 Franz Warner Parkway
 Whitsett, N.C. 27377-3000
 USA
 T +1-336 4 49 44 40
 F +1-888 5 70 45 54
 info@mediusa.com
 www.mediusa.com

M.4s PCL dynamic

Knieorthese Knee Brace Orthèse pur le genou Ortesis de rodilla

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию. Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Használati útmutató. Uputstvo za upotrebu. Інструкція з використання. 使用说明.

medi

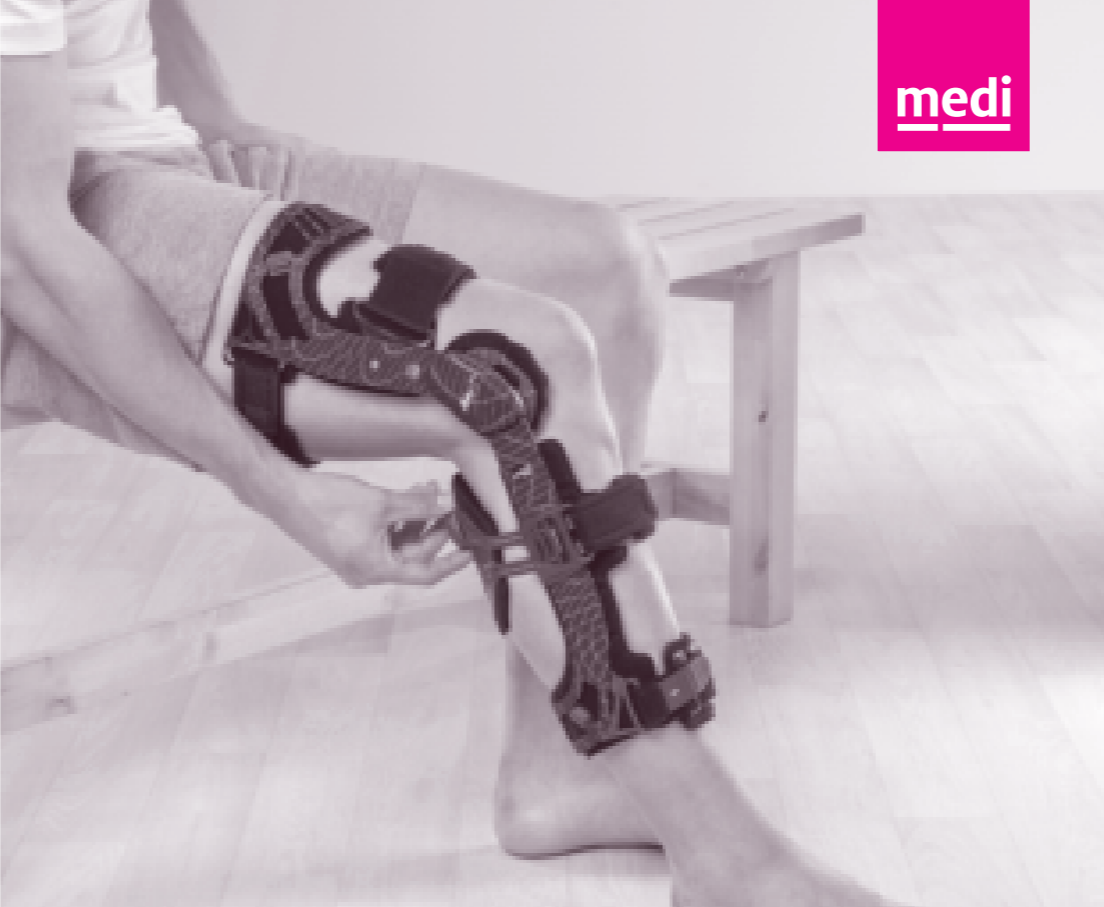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Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. It is possible that you may feel pain in your calf when you first start wearing the brace. Please contact your physician or orthopaedic technician immediately if the pain doesn't get considerably better or if you suffer excessive pain while wearing the brace. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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M.4s PCL dynamic

Intended purpose

M.4s PCL dynamic is a functional knee orthosis. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin or if any wounds have been properly covered.

Indications

- Conservative treatment following injury to the posterior cruciate ligament
- Post-operative treatment following reconstruction of the posterior cruciate ligament

Contra-indications

- None known at present

Applying the brace

Before use:

- Open all straps and pull them through the belt eyes. In order to prevent the the Velcro® from causing damage to clothing, for example, fold the end of the strap back on to it.
- Pull the knob out of the tibia support (Fig. 1). A click confirms the correct position. Pull the tibia support out towards the back until it stops (Fig. 2).

Putting the brace on:

- Bend your leg to -70° and put the brace on the injured leg (Fig. 3).
- Position the joints in such a way that the centre of the joint (extension wedge level) is located at the top edge of the kneecap. Please make sure that the brace has been placed on the leg straight and is not twisted. (Fig. 4)
- Fasten Belt No. 1 first (Fig. 5) and then Belt No. 2 (Fig. 6)
- Keep your leg bent and check again

that you have put the brace on properly.

- The brace has been put on properly if the joint centres are located at the top edge of the kneecap and they are positioned slightly behind the side central line of the leg. (Fig. 7).
- Now fasten Belt No. 3 for the thigh and pull it tight. Make sure that you don't position the brace too far forwards. (Fig. 8+9).
- Turn the knob back in to activate the turning mechanism. Another click will confirm the correct position (Fig. 10).
- Now turn the wheel to the extent that the tuberosity of the tibia (protruding bone at the top end of the shin) is level with the one on the healthy leg (Fig. 11 + 12).
- When required and for purely stabilising purposes, please fasten the tibia belt Δ . This must always be the last belt to be fastened (Fig. 13).
- Now move the knee lightly a few times and check again that you have put the brace on properly. Make any necessary corrections (Fig. 14)
- The indicators on the side on the guide bars may be used as a reference point for the position of the tibia support.

Setting the flexion / extension limits

Extension limit wedge, silver:

$0^\circ, 10^\circ, 20^\circ, 30^\circ, 45^\circ$

Flexion limit wedge, black:

$0^\circ, 10^\circ, 20^\circ, 30^\circ, 45^\circ, 60^\circ, 75^\circ, 90^\circ$

Can be immobilised at:

$0^\circ, 10^\circ, 20^\circ, 30^\circ, 45^\circ$

For immobilising the orthosis first insert the extension wedge and then the corresponding flexion wedge. The medial (inside) and the lateral (outside) wedge settings must always be the same.

1. 0° extension and 90° flexion wedges are pre-installed in your knee brace as standard (In the UK only 10° extension wedges fitted, remainder included as spare parts).

2. In order to change the wedges, press the joint cover on the rear side (see profile) outwards.
3. You can change the wedges by removing the tweezers from the joint cover and pulling the wedges forwards (extension) or backwards (flexion) out of the joint using the tweezers. The wedges are accordingly labelled with „FLEX“ or „EXT“ and the corresponding degree. The wedges may also be distinguished by the colour. Extension wedges are silver, flexion wedges are black. Push the new wedge from the front side (extension) or from the back (flexion) into the joint, so the markings „UP“ on the joint and the wedge are matched accordingly. Upon proper locking you will hear a click sound.
4. Move the joints of the orthosis to check the position of the wedges. Put the tweezers back into the mounting and attach the joint covers by hooking the two noses of the joint cover into the front side of the joint and then pushing the cover onto the joint. The cover is firmly attached if you hear a click sound.

Important notes

Only change the extension and flexion limits on instruction from your doctor or orthotist. In order to prevent hyperextension, always insert extension wedges or 0° wedges. The wedges define the exact excursion of the orthosis. In difficult soft-tissue conditions, e.g. where the soft-tissues are under compression, it may be necessary to insert the larger wedge to limit the joint movement to the desired excursion. In addition, make sure that the orthosis joints are correctly positioned: the middles of the joints should be level with the upper border of the patella. The joint must lie behind the midline of the leg.

Shaping the orthosis

When shaping the orthosis, please ensure that the joints stay parallel. This guarantees that the orthosis will function well and run easily and that there is no excessive wear.

Care instructions

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

With normal use, the hinge mechanism should not require any maintenance, but it can be lubricated with Teflon spray. Check regularly that the joint screws are secure, tighten them if necessary.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Avional, Polyamide, PU, Polyester

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

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